

### Scanner components:

① Scanner body ② USB 3.0 Repeater ③ Power adapter


⑦ Cradle ⑧ Scanner tip ⑨ Dust cap

### Calibrator components:

④ Calibrator ⑤ USB 3.0 cable ⑩ Dust cover

### Other devices:

⑥ PC ⑪ USB flash drive

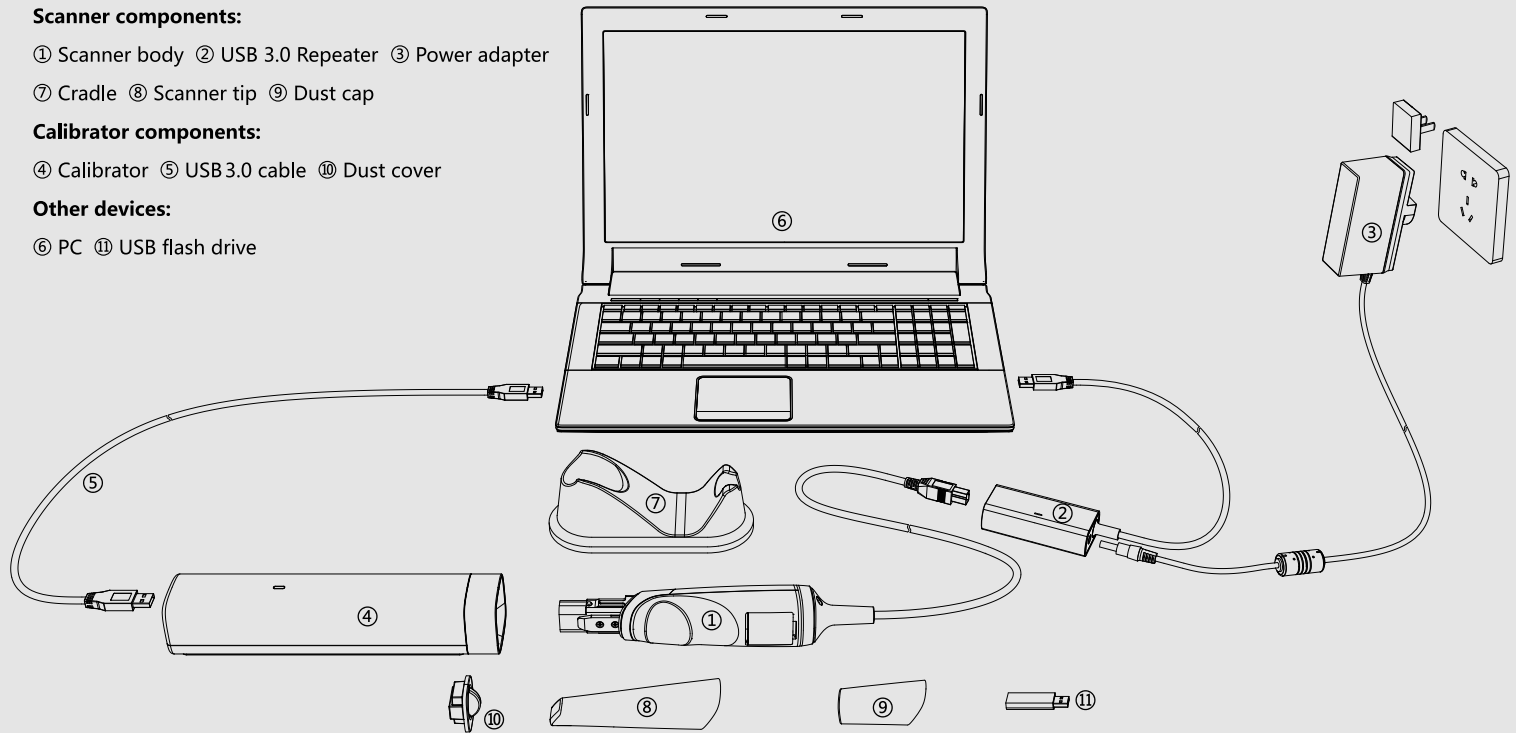

### Installation Steps

1. Install the scanner after installing the software. Follow the sequence number above to install the items from ① to ⑥. Connect the adapter to the power.
2. Make sure that the indicator lights on scanner, USB 3.0 Repeater and calibrator are green.
3. Plug the scanner into the calibrator, there will be a sound "click" and open the software to calibrate the scanner.
4. Remove the calibrator after calibration. Plug the tip into the scanner to scan.

### Cautions

- During the calibration, the scanner should be fully inserted into the calibrator, and should be placed at a horizontal plane.
  - To keep the inner components clean, the dust cover should be installed when the calibrator is not used.
  - Connect the USB 3.0 Repeater to the PC USB 3.0 port.
-