

# SYNTEX

Root canal filling and sealing material

CE<sup>1011</sup>

BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY  
PRODUCT FOR DENTAL USE ONLY

COMPOSITION: epoxy resin, calcium tungstate, zirconium oxide, amines, silicon oxide, silicon oil.

## INTENDED USE

Permanent root canal filling of the permanent teeth by means of gutta-percha points.

Permanent root canal filling of the permanent teeth by means of root canal points. The material can be used for all techniques of the root canal sealing. It is perfect for thermic methods.

## INSTRUCTIONS FOR USE

Sealing paste SYNTEX occurs in two self mixing syringes, dispensing two pastes in the proportion 1:1.

Prepare, rinse and dry the root canal prior to application.

Caution! You must use the original self mixing valve attached with the product in order to properly mix the components.

Place a small amount of SYNTEX on the tip of Lentulo needle; slowly move Lentulo needle towards the apex at the lowest possible micro-engine speed. Remove Lentulo from the canal using the minimal engine speed. Apply SYNTEX paste to a previously disinfected and dried point. Place it in the canal with a pumping movement. Additional points may be placed in the canal or it may be filled with the method of lateral or vertical condensation of gutta-percha.

## CONTRAINDICATIONS

Do not use SYNTEX in patients with hypersensitivity to the product components.

Avoid contact of the non-polymerised material with skin or mucous membranes.

In case of contact, immediately flush with large amounts of water.

## PRECAUTIONS

Use a dental dam while working with the product.

Use protective gloves, goggles and clothes.

Causes skin irritation. Harmful if swallowed. Toxic to aquatic organisms.

Irritating to oral mucosa. In case of contact, immediately flush with water.

The product is irritating to eyes. In case of eye contact, immediately flush with large amounts of water.

Do not induce vomiting if swallowed. Drink large amounts of water. Contact a physician.

## STORAGE

Store in a sealed package in a ventilated room, at the temperature of 2-25°C. Protect from sunlight.

Keep out of reach of children.

The use-by date is placed on the product label.

## DISPOSING OF PACKAGES

Used packages should be submitted for disposal or returned to the manufacturer.

## PACKAGING

A self mixing syringe containing 10 g of the product.



PPH CERKAMED Wojciech Pawłowski

37-450 Stalowa Wola, Polska

ul. Kwiatkowskiego 1

Tel/fax 00 48 15 842 35 85

Last update: 20.04.2015/2