Bond-1™ SF

Solvent-Free SE Adhesive

English

INSTRUCTIONS FOR USE

Français

French

MODE D'EMPLOI

Deutsch

German

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Italiano

Italian

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Español

Spanish

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Čeština

Czech

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English

INSTRUCTIONS FOR USE:

Bond-1™ SF Solvent Free SE Adhesive Instructions For Use

Bond-1™ SF Solvent Free SE Adhesive is a solvent free, single component, self-etching, light cure bonding agent. By removing the solvent, such as acetone, alcohol, water, etc., it eliminates the need to air dry, thus virtually eliminating sensitivity and saving precious chair time for both the patient and the clinician. Furthermore, the adhesive layer formed eliminates “water treeing” and will maintain the integrity of the bonding interfaces. Bond-1 SF Solvent Free SE Adhesive is designed mainly for direct composite restorations and demonstrates high bond strengths to both dentin and enamel.

INDICATIONS:

- Direct composite restorations
- Bonding agent for repairs to composite, ceramic and metal substrates
- Pit and fissure sealant
- Compatible with dual cure materials, only when the initial layer is light initiated

CONTRAINDICATIONS:

- Not to be used on patients with known allergies to methacrylate resins
- Not to be used with self-cure only materials
- Not for use on exposed pulp
- Not recommended on uncut enamel

Storage and Handling:

Store at 2°–8°C (36°–46°F). Use at room temperature.

Instructions

1. Prepare tooth according to principles of adhesive dentistry. Make sure to clean and remove surface debris thoroughly from the preparation site.
2. Use oil-free air to dry the tooth for 2-3 seconds or use an absorbent pellet to blot tooth dry. There should be no visible water on the tooth's surface, but do not over dry the tooth.
3. **Single Dose:** Twist the top of the single dose container until the top snaps off. Dip the microbrush into the single dose, wetting the microbrush thoroughly.

Syringe: Remove cap and place a 25g flocked needle tip onto the syringe. Be sure tip is securely in

- place. Dispense and spread the material slowly, through the needle tip, against the tooth surface.
4. Apply a layer of the bonding agent to the entire tooth surface, using a rubbing motion to distribute the material evenly and thoroughly.
 5. Using the needle tip or single dose microbrush, continue to scrub the material on the tooth's surface for 20 seconds.
 6. Light cure for 10 seconds.
 7. Apply composite according to manufacturer's instructions.
 8. Replace cap and discard needle tip after use. Single Dose should be discarded after use.

Note: The syringe should be disinfected after each use with disinfectant wipes or disinfecting solution. Do not submerge syringe in disinfecting solution.

Bond-1 SF is offered in syringe and single dose delivery.

NOTE: Single dose delivery, tips, and applicator brushes are for single patient use only, to prevent cross contamination between patients.

NOTE: Refer to outer package for expiration date. Do not use after expiration date.

CAUTION: Uncured methacrylate resin may cause contact dermatitis and damage the pulp. Avoid contact with skin, eyes and soft tissue. Wash thoroughly with water after contact.

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Solvent-Free SE Adhesive

Rx Only

Refrigeration required.
Store at 2-8°C. (36-46°F)

PENTRON

Manufactured by Pentron Clinical

1717 West Collins Ave. Orange, CA 92867 USA
800-551-0283 / 714-516-7557 / www.pentron.com

 European Union Representative

SpofaDental a.s.

Markova 238

CZ-506 01 Jičín

Phone: +420 493 583 111

LIMITED WARRANTY –

Pentron's technical advice, whether verbal or in writing, is designed to assist dentists in using Pentron's product. The dentist assumes all risk and liability for damages arising out of the improper use of Pentron's product.

In the event of a defect in material or workmanship, Pentron will replace the defective product or part thereof, or reimburse the actual cost of the defective product. In order to take advantage of this limited warranty, the defective product must be returned to Pentron.